

SYRINGE DESTROYER – BD-300C



CODE: SYRDST

The disposable syringe destroyer is easy to operate and safe and only requires 6V of working voltage to protect patients from infection from used syringes.

Features:

- Power: 220V/110V AC, 50/60Hz, <130W
- The equipment uses the silver-pole with low voltage (about 7 V.A.C) to melt the needle conveniently and quickly within three seconds
- No pollution, only produces a small amount of waste

To Operate: First turn on the power, squeeze the residue liquid out from the syringe body, put the needle directly into the needle melting hole firstly for melting the needle, and then, press the syringe body downward directly until a hole is punctured into the syringe.